

Mepilex® Border Lite

THIN, ABSORBENT, SELF-ADHERENT FOAM DRESSING WITH SAFETAC

- Atraumatic to the wound and surrounding skin on removal
- Thin and conformable all-in-one foam dressing with Safetac soft silicone technology
- Seals around the wound margins to reduce the risk of maceration

Film backing with dynamic permeability

- Viral proof, bacterial proof and waterproof to protect the patient and the clinician
- Maintains a moist wound environment
- Showerproof

Safetac® soft silicone layer

- Self-adherent soft silicone layer
- Does not adhere to the moist wound bed
- Adheres gently to the surrounding skin
- Minimizes trauma and pain on removal



Absorbent foam pad


- Absorbs exudate
- Multi-layered construction to optimize fluid handling and prevent leakage
- Maintains integrity
- Conforms to body contours

Safetac
TECHNOLOGY

Safetac® technology

Safetac® technology. Less trauma. Less pain.™

Safetac is a patented soft silicone adhesive technology. Dressings with Safetac technology are atraumatic both during wear and upon removal. These dressings minimize trauma to the wound and the surrounding skin, which minimizes pain to the patient. They also prevent maceration by sealing around the margin of the wound to protect peri-wound skin.

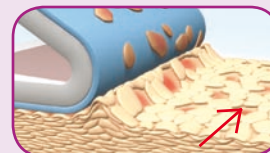

Skin stripping occurs with traditional adhesive




No skin stripping occurs with Safetac


How Mepilex Border Lite works

Mepilex Border Lite is a highly conformable, thin, soft silicone foam dressing that absorbs small amounts of exudate and maintains a moist wound environment. It adheres gently to surrounding skin but doesn't adhere to moist wound tissue, thereby minimizing trauma to the wound and peri-wound area and pain to the patient.

How to use Mepilex Border Lite

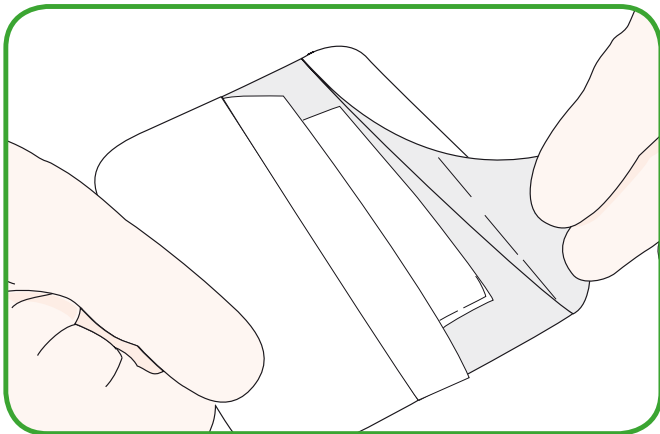

1. Gently cleanse the wound; dry surrounding skin. Choose a dressing size that will allow the adherent border to overlap the wound area by at least 2cm. Remove the release film.

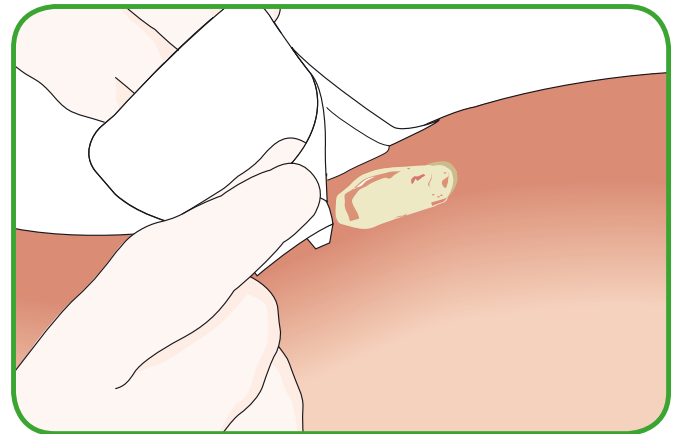

2. Lay absorbent foam pad over wound area. Gently smooth adherent border in place. Do not stretch. Mepilex Border Lite can remain in place for up to 7 days depending on drainage and wound condition.

Benefits of Mepilex Border Lite

- Minimizes trauma and pain during dressing changes
- All-in-one dressing
- Thin, conformable and absorbent
- Viral/bacterial/waterproof to protect patient and clinician
- Can remain in place for up to 7 days
- Minimizes the risk of maceration

Mepilex Border Lite



Mepilex Border Lite Assortment

Art. no	Size cm	Pcs/box	Pcs/case
281000	4 x 5	10	70
281100	5 x 12.5	5	65
281200	7.5 x 7.5	5	70
281300	10 x 10	5	50
281500	15 x 15	5	50