Mepilex[®] Lite

Thin, absorbant foam dressing with Safetac[®] soft silicone

Thin foam pad

- Absorbs exudate
- Conforms to body contours
- Maintains integrity

Safetac[®] soft silicone layer

- Non-adherent to moist wound bed
- Adheres gently to surrounding skin
- Minimizes trauma and pain on removal
- Protects against the risk of maceration

Film backing

- Moisture vapour permeable
- Waterproof
- Maintains a moist wound environment



- Atraumatic to the wound and surrounding skin on removal
- Thin, conformable and absorbent
- Seals around the wound margins to reduce the risk of maceration

Proven choice for a better outcome

Safetac*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

* A unique proprietary technology exclusive to Mölnlycke Health Care

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex[®], Mepitel[®], Mepiform[®] and Mepitac[®]. To date, these dressings have helped millions of patients worldwide⁷⁻⁹.





Mepilex[®] Lite

Directions for use



1. Gently cleanse the wound; dry surrounding skin. Choose a dressing size that will allow the Mepilex[®] Lite dressing to overlap the wound area by 2 cm.



2. Remove one side of the release film and apply Mepilex[®] Lite to the side of wound. Do not stretch. Remove remaining film and gently smooth in place.



3. Border with Mepitac[®] soft silicone tape to fixate in place or cover with a non-adhesive fixation product such as Tubifast[®].

How Mepilex[®] Lite works

Mepilex[®] Lite is a highly conformable, thin, soft silicone foam dressing that absorbs small amounts of exudate and maintains a moist wound environment. The Safetac[®] layer seals around the wound edges to protect against the risk of maceration and ensure atraumatic dressing changes. Mepilex[®] Lite can be cut to size if required.

Benefits of Mepilex® Lite

- Atraumatic to the wound and surrounding skin on removal
- Minimizes trauma and pain related to dressing change
- Minimizes the risk of maceration
- Conforms well to body contours
- Thin and comfortable to wear
- Easy to use
- Maintains a moist wound environment

Indications for use

Mepilex[®] Lite is designed for the management of a wide range of non to low exuding wounds, such as leg and foot ulcers, pressure ulcers, partial thickness burns, radiation skin reactions and Epidermolysis Bullosa. Mepilex[®] Lite can also be used as protection of compromised and/or fragile skin. Mepilex[®] Lite can be used under compression and in conjunction with hydrogels.





Mepilex[®] Lite Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
284000	6 x 8.5	5	70
284100	10 x 10	5	50
284300	15 x 15	5	50
284500	20 x 50	4	24

Mölnlycke 1-800-494-5134 www.molnlycke.ca

Molnlycke Health Care AB, Box 13080, SE-40252 Göteborg, Sweden. Phone +46317223000. The Mölnlycke Health Care, Mepilex® Lite, and Safetac® names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. © 2018 Mölnlycke Health Care AB. All rights reserved.