

# Mepore®

Self-adhesive, absorbent  
post-operative dressing

- Two part release film for easy application

- Skin-friendly, water based, solvent-free polyacrylate adhesive for gentle and secure fixation



- Soft non-woven spunlace polyester fabric with high breathability for enhanced patient comfort

- Absorbent wound pad with low adherent contact layer for enhanced patient comfort and longer wear time.

- Mepore® is an air permeable, self-adhesive and absorbent dressing designed especially for surgical wounds. Its skin friendly, water-based, solvent-free polyacrylate adhesive provides a secure yet gentle fixation. Mepore® is soft, conformable, water repellent and radioparent.

Mepore®

  
Mölnlycke®

## Directions for use



**1.** Remove from package and remove outer half of release film.



**2.** Gently fixate to skin at one end of incision site. Position dressing over incision site. DO NOT STRETCH.



**3.** Remove the remaining half of the release liner and smooth adhesive border to the skin.

### Frequency of change

Change Mepore® as indicated by wound condition or existing protocol. To remove, loosen one edge and slowly peel the dressing from the skin in the direction of hair growth.

## How Mepore® works

The porous structure of the nonwoven fabric ensures air and water vapour permeability. The fabric is flexible and conforms well to body contours. The fabric is non-absorbent and provides a water-repellent barrier for blood and fluid strike-through. The polyacrylate adhesive has been designed to be gentle to the skin while at the same time providing secure fixation. The wound pad provides ideal absorption for sutured surgical wounds and features a low adherent wound contact layer. Since Mepore® is radiopaque, it can remain in place during x-ray.

### Benefits of Mepore®

- Gentle skin-friendly, water-based adhesive
- Absorbent, non-adherent wound pad keeps wound clean, reduces the risk of wound contamination, minimizes the risk of adherence to the wound and reduces dressing change frequency
- Flexible, fluid-repellent backing
- Air permeable to prevent maceration and promote comfort
- Rounded corners for secure fixation (all sizes up to 9x15cm)

### Mepore® Assortment (Sterile packed)

Art. no	Size cm	Pcs/Box	Pcs/Case
670800	6 x 7	60	480
670900	9 x 10	50	400
671000	9 x 15	50	400
671100	9 x 20	30	180
671200	9 x 25	30	210
671300	9 x 30	30	270
671500	10 x 11	40	280

### Mepore® Non-Sterile Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
331900	4 cm x 5 m	1	24
332000	7 cm x 5 m	1	18

Mepore® non-sterile is packaged in 5 metre rolls.

Mölnlycke  
1-800-494-5134  
www.molnlycke.ca

Mölnlycke Health Care AB, Box 13080, SE-40252 Göteborg, Sweden.  
Phone +46317223000. The Mölnlycke Health Care and Mepore® names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies.  
© 2018 Mölnlycke Health Care AB. All rights reserved.

## Indications for use

Mepore® is appropriate for wounds with low to moderate exudate levels. These include surgical incisions, sutured wounds, Laparoscopy and Arthroscopy puncture sites, abrasions, traumatic wounds and cannula sites.

## Precautions

Care should be taken to ensure that Mepore® is not applied under tension, to prevent shearing force which can cause damage to the skin. This is particularly important when the dressing is applied over joints.



  
**Mölnlycke®**

CEWC01011805