

Mepilex® Border Lite


Thin, self-adherent, absorbent foam dressing with Safetac® technology

Backing film with dynamic permeability

- Viral proof, bacterial proof and waterproof to protect the patient and the clinician
- Maintains a moist wound environment
- Showerproof

Safetac® soft silicone layer

- Self-adherent soft silicone layer
- Does not adhere to the moist wound bed
- Adheres gently to the surrounding skin
- Minimizes trauma and pain on removal

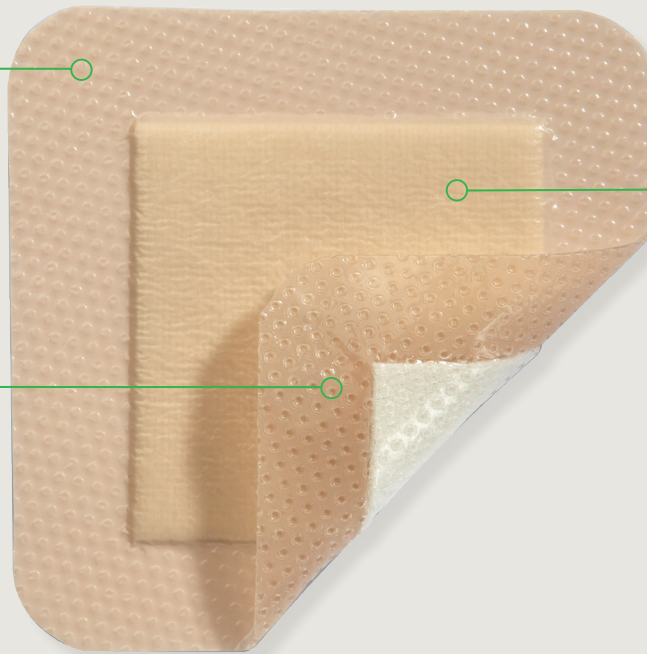

Absorbent foam pad

- Absorbs exudate
- Multi-layered construction to optimize fluid handling and prevent leakage
- Maintains integrity
- Conforms to body contours

Safetac
TECHNOLOGY

- Atraumatic to the wound and surrounding skin on removal
- Thin and conformable all-in-one foam dressing with Safetac® soft silicone technology
- Seals around the wound margins to reduce the risk of maceration

Proven choice for a better outcome

Safetac*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex®, Mepitel®, Mepiform® and Mepitac®. To date, these dressings have helped millions of patients worldwide⁷⁻⁹.


* A unique proprietary technology exclusive to Mölnlycke Health Care

Safetac
TECHNOLOGY


Mepilex® Border Lite


Mölnlycke®

Directions for use

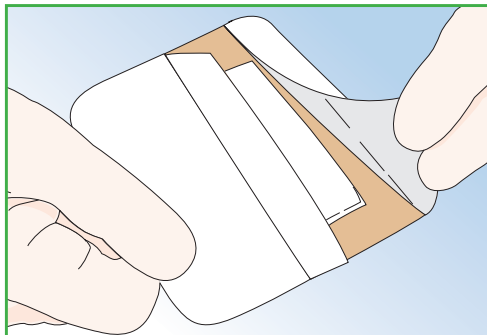

1. Gently cleanse the wound; dry surrounding skin. Choose a dressing size that will allow the adherent border to overlap the wound area by at least 2cm. Remove the release film.

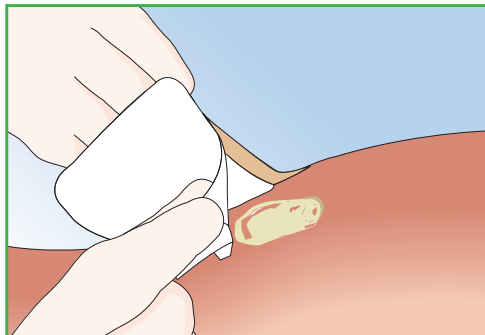

2. Lay absorbent foam pad over wound area. Gently smooth adherent border in place. Do not stretch. Mepilex® Border can remain in place for up to 7 days depending on drainage and wound condition.

How Mepilex® Border Lite works

Mepilex® Border Lite is a highly conformable, thin, soft silicone foam dressing that absorbs small amounts of exudate and maintains a moist wound environment. It adheres gently to surrounding skin but doesn't adhere to moist wound tissue, thereby minimizing trauma to the wound and peri-wound area and pain to the patient.

Benefits of Mepilex® Border Lite

- Minimizes trauma and pain during dressing changes
- All-in-one dressing
- Thin, conformable and absorbent
- Viral/bacterial/waterproof to protect patient and clinician
- Can remain in place for up to 7 days
- Minimizes the risk of maceration

Indications*

Mepilex Border Lite is designed for the management of a wide range of non/low exuding wounds, such as leg and foot ulcers, pressure ulcers and traumatic wounds, e.g. abrasion, finger injuries, blisters and skin tears.

Mepilex Border Lite can also be used as protection of compromised and/or fragile skin.

Precautions*

Do not use on patients with known sensitivity to the dressing or its components.

In the case of signs of clinical infection, consult a health care professional for adequate infection treatment.

Do not use Mepilex Border Lite together with oxidizing agents such as hypochlorite solutions or hydrogen peroxide.

Mepilex® Border Lite Assortment (Sterile packed)

Art. no	Size cm	Pcs/Box	Pcs/Case
281000	4 x 5	10	70
281100	5 x 12.5	5	65
281200	7.5 x 7.5	5	70
281300	10 x 10	5	50
281500	15 x 15	5	50

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

