

Mepilex® Transfer


Effective transfer dressing with Safetac® soft silicone

Ultra thin foam 'exudate transfer' dressing

- Protects the wound bed while transferring exudate vertically, away from the wound and into an absorbent cover dressing

Safetac® soft silicone layer

- Non-adherent to moist wound bed
- Soft silicone adheres gently to the surrounding skin
- Ensures removal with minimal trauma and pain
- Prevents maceration

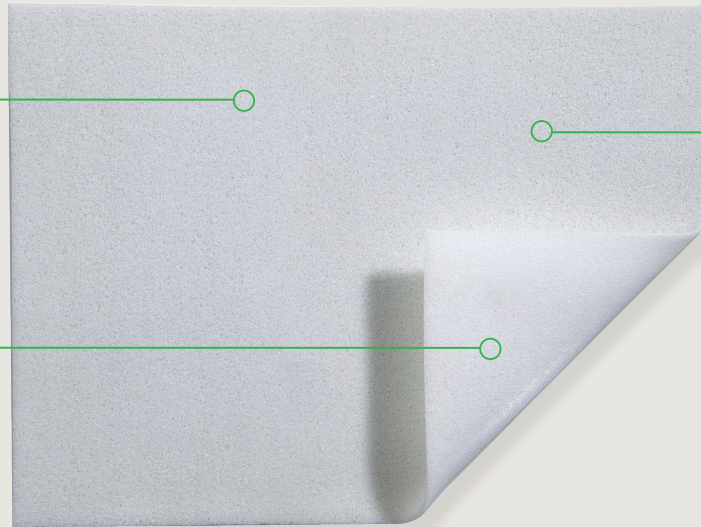

Conforms well

- Conforms well even on difficult-to-dress wounds
- Suitable for large, fragile wounds and difficult to dress areas

Safetac
TECHNOLOGY

- Transfers exudate away from the wound and minimizes the risk of maceration
- Thin and highly conformable for difficult-to-dress wounds

- Atraumatic to the wound and surrounding skin
- Can remain in place for up to 7 days which promotes cost-effective wound healing

Proven choice for a better outcome

Safetac*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex®, Mepitel®, Mepiform® and Mepitac®. To date, these dressings have helped millions of patients worldwide⁷⁻⁹.


* A unique proprietary technology exclusive to Mölnlycke Health Care

Safetac
TECHNOLOGY


Mepilex® Transfer


Mölnlycke®


Directions for use

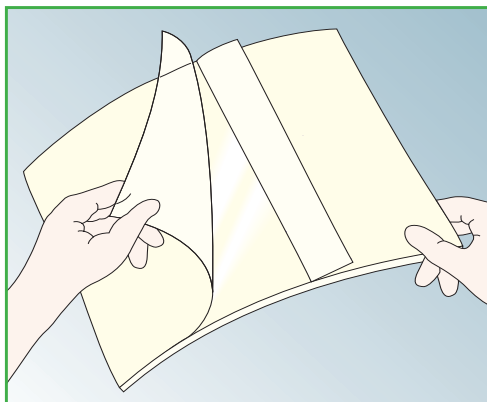

1. Cleanse the wound area and dry the surrounding skin. Remove half of the release film.

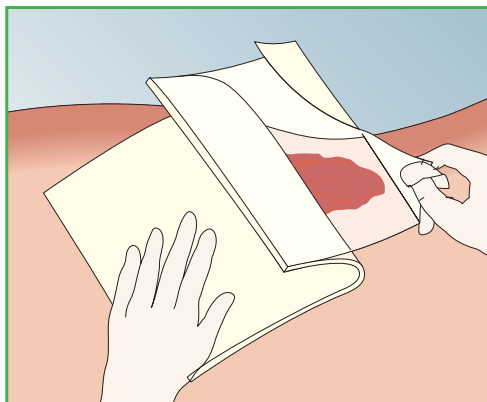

2. Apply one side of dressing allowing Mepilex® Transfer to overlap the surrounding skin by at least 5 cm. Remove remaining release film and smooth in place.

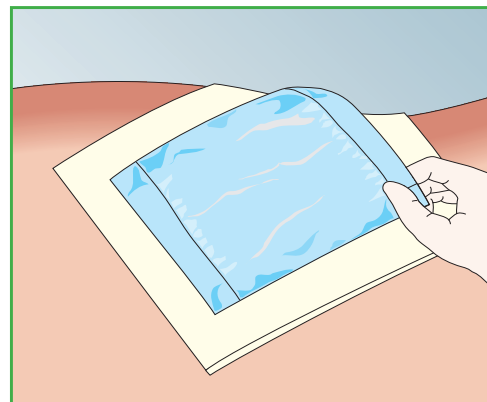

3. Apply an appropriate cover dressing according to wound exudate level and fixate in place.

How Mepilex® Transfer works

Mepilex® Transfer is thin and conformable which, in combination with the Safetac® soft silicone layer ensures direct contact to the wound and the surrounding skin even on irregular surfaces. As the Safetac® layer seals around the wound, the foam structure of Mepilex® Transfer allows the exudate to move vertically into a secondary absorbent pad. Mepilex® Transfer, in combination with an appropriate secondary dressing, maintains a moist wound environment.

Benefits of Mepilex® Transfer

- Transfers exudate away from the wound
- Conforms well to body contours
- Can be cut to size if required
- Promotes patient comfort during wear
- Can be used under compression
- Minimizes trauma and pain

Mepilex® Transfer Assortment

Art. no	Size cm	Pcs/Box	Pcs/Case
294600	7.5 x 8.5	5	70
294700	10 x 12	5	50
294800	15 x 20	5	40
294502	20 x 50	4	24

*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to www.molnlycke.ca

Indications*

Mepilex® Transfer is designed for a wide range of exuding and difficult-to-dress wounds.

Mepilex® Transfer can also be used as a protective layer on non-exuding wounds and/or large areas of fragile skin. Mepilex® Transfer can be used under compression.

Note:

Mepilex® Transfer can remain in place for up to 7 days as indicated by wound condition. The absorbent cover dressing may be changed more frequently if required.

Precautions*

- Do not use on patients with known sensitivity to the dressing or its components.
- In the case of signs of clinical infection, consult a health care professional for adequate infection treatment.
- Do not use Mepilex® Transfer together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.

