

# Mestopore® S

Absorbent continent stoma dressing  
with soft silicone adhesive

- Safetac® soft silicone adhesive technology for gentle adherence to dry skin and removal with less trauma and less pain

- Breathable nonwoven film, impermeable to fluids for comfort and protection



- Low-adherent absorbent pad with no fibre linting for excellent fluid management and added security

- Mestopore® S is a self-adhesive absorbent dressing with soft silicone adhesive for continent stomas. Mestopore® S gives less trauma to the surrounding skin and less pain to the patient during dressing changes.

## Proven choice for a better outcome

Safetac\*, pioneered by Mölnlycke, delivers above and beyond the ordinary. Proven to help optimize the wound healing journey and even prevent wounds, dressings with Safetac are the safe choice for patients and a champion for higher standards in wound care.

In fact, we have a wealth of evidence that supports the clinical and economic benefits of dressings with Safetac, including Mepilex®, Mepitel®, Mepiform® and Mepitac®. To date, these dressings have helped millions of patients worldwide<sup>7-9</sup>.


\* A unique proprietary technology exclusive to Mölnlycke Health Care

**Safetac**  
TECHNOLOGY


Mestopore® S

  
**Mölnlycke**<sup>®</sup>


## Directions for use

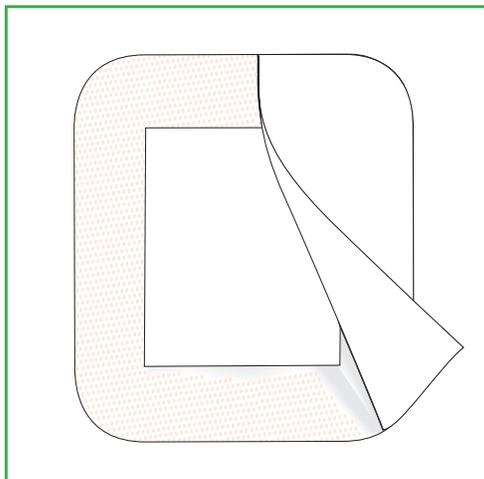

1. Remove "release film" from bottom half of Mestopore® S.

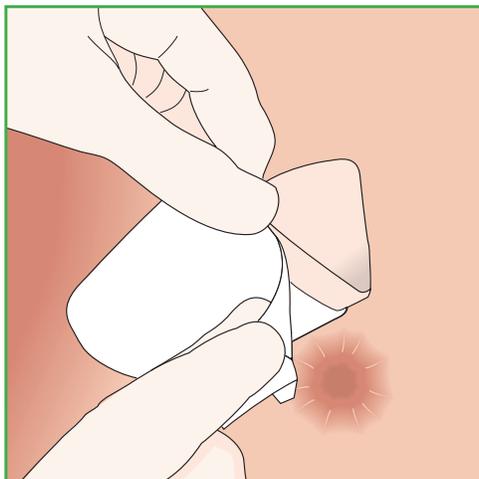

2. Apply to skin under stoma, then lift dressing upward over stoma. Remove 'release film' from top half of Mestopore® S, press the dressing gently against skin above stoma.

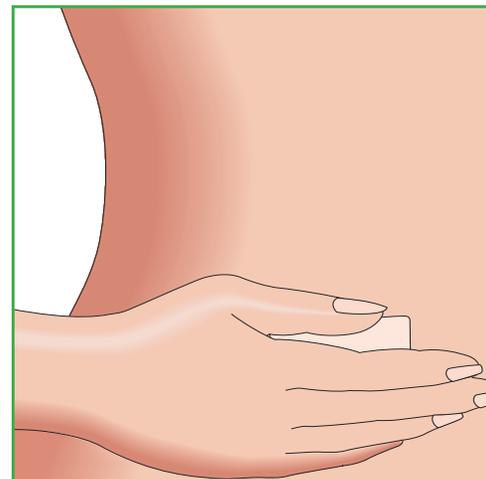

3. Gently smooth adhesive border against the skin to ensure secure fixation. To remove dressing, gently lift from "side-to-side" (not "top-to-bottom").

## How Mestopore® S works

Mestopore® S is a self-adherent, absorbent dressing with super-absorbent fibres and Safetac® soft silicone microadhesive technology. A non-adherent contact layer protects the stoma tissue while the highly absorbent "superabsorbent fibres" absorb and retain large amounts of fluid and secretions. The self-adherent border prevents leakage and is non-irritating, even with frequent application and removal. The outer film material is moisture vapour permeable and water-proof for optimum comfort and security. Mestopore® S can be lifted and re-applied to accommodate irrigation or inspection of stoma site.

## Benefits of Mestopore® S

- Minimizes pain and tissue damage during dressing changes
- Non-sensitizing
- Adapts well to body contours
- Shower-proof
- No residue
- Easy to use

## Mestopore® S Assortment

Art. no	Size cm	Pcs/Box
665100	10 x 10	20

### Note

Mestopore® is also available with a traditional adhesive border.

- Water-based, non-sensitizing polyacrylate adhesive for gentle and secure fixation.

## Mestopore® Assortment (with traditional adhesive)

Art. no	Size cm	Pcs/Box
665000	9 x 10	8

## Indications for use

- Continent Stomas
- Mucous fistula

## Precautions

In case of clinical signs of infection, consult a health care professional for adequate infection treatment.



\*Notice: For Mölnlycke licensed product details including indications and precautions, please refer to [www.molnlycke.ca](http://www.molnlycke.ca)